Simplex

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Dear Customer,

Thank you for choosing our product and for the trust you have placed in us. In order for you and the user to enjoy your new product safely, conveniently, and comfortably, we kindly ask that you read the Instructions for Use. Doing this will also help avoid damage, injury, and incorrect operation and handling of the product. Please ensure that a copy of the Instructions for Use remains with the product so that every user has access to it.

If you have any questions, please contact your authorised distributor or us directly.

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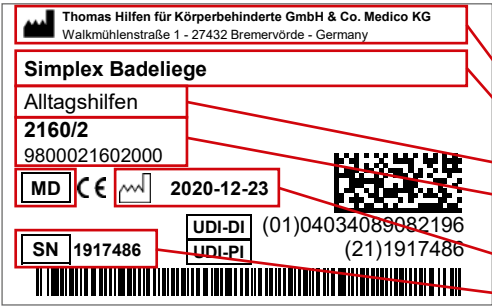
We reserve the right to make technical changes. This can lead to differences between your product and how it is presented in brochures and other media.

Table of Contents

Figures	2
Preface / Manufacturer	12
Labels / Model Identification	13
General Remarks / Safety Instructions	14
Intended Purpose	15
– Intended Use / Application Areas	15
– Indication / Patient Target Group / Intended Users	15
– Risks Associated with Use and Contraindications	15
Product Description	15
– Scope of Delivery / Standard Equipment	15
– Putting into Service	16
– Teaching the User	16
Adjustment Options / Features	16
– Technical Data	16
– Material Specifications	17
Cleaning, Maintenance, and Repair	17
– Cleaning, Care Instructions, and Disinfection on a Daily Basis	17

– Maintenance and inspection by the distributor / Maintenance Schedule . . .	17
– Spare Parts	18
– Storage / Disposal	18
EU Declaration of Conformity	18
Warranty	19
Period of Use / Service Life	19
Reuse	19
Distributor	19

Labels / Model Identification

Label	Meaning
 <p>The image shows a detailed barcode label with the following information:</p> <ul style="list-style-type: none"> Manufacturer: Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG, Walkmühlenstraße 1 - 27432 Bremervörde - Germany Product Name: Simplex Badeliege Product Group: Alltagshilfen Item Number short / long: 2160/2, 9800021602000 MD = Medical Device marking Manufacture Date: 2020-12-23 UDI-DI (01)04034089082196 UDI-PI (21)1917486 Serial Number: SN 1917486 	<p>Barcode Label (Detail)</p> <p>Position on the Product (Fig. 1)</p> <ul style="list-style-type: none"> - Manufacturer - Product Name - Product Group - Item Number short / long - MD = Medical Device - Manufacture Date - Serial Number
 <p>The image shows a type plate with the following information:</p> <ul style="list-style-type: none"> Manufacturer: Thomashilfen, Walkmühlenstraße 1, 27432 Bremervörde, Germany Product Name: Simplex max. user weight: max. Nutzergewicht / max. user weight: 25 kg / 55.1 lbs CE marking Information icon (i) 	<p>Type plate</p> <p>Position on the Product (Fig. 1)</p> <ul style="list-style-type: none"> - Manufacturer - Product Name - Max. user weight - CE marking - Follow Instructions for Use

Remarks

General Remarks

- Read the Instructions for Use carefully before first use. If you have difficulty reading, please have them read to you. If the copy of the Instructions for Use is missing, you can download one from www.thomashilfen.de. The contents of the digital Instructions for Use can be enlarged.
- Check the packaging for any transport damage and to be sure the product is in flawless condition.
- Changes to the product are only permitted within the scope of the intended adjustment options.
- Cleaning and daily care can be provided by relatives and caregivers. Maintenance and inspection are done by the distributor.
- Clean the frame of the product as needed.
- Always wipe the product dry if it gets wet. Never put away wet after use.
- All labels have to remain on the product and should not be tampered with.

Safety Instructions

- Check at least every 3 months that the settings on the product still work for the user's current body measurements. If need be, have a distributor / orthopaedic technician adjust the settings.
- Do regular (weekly) checks to make sure that the setting options on the product work.
- Only use the bath chair lift in fixed bath tubs.
- Be careful of the moving parts when adjusting or folding the product.
- Only lift up the product by its firmly welded or screwed parts.
- Before each use check that all the nuts, bolts, and screws are tight.
- Before you use the chair lift, check the connecting cord to lower the chair lift for damage.
- Please do not overload the product and note the maximum user weight and the maximum load capacity (see technical data).
- Never leave the user unattended with the product.

- The product should only be used by one user at a time. No other individuals should be placed in or on the product.
- All users have to be familiar with how the product works. This way in an emergency the user can be removed quickly from the product.
- If there is a serious and unexpected accident with the product, please inform our company immediately and the responsible authorities.

Intended Purpose

Intended Use / Application Areas

The Simplex bathing chair is a class 1 medical device.

The bath chair lift is suitable for use in the bathtub only. The edge of the bathtub has to provide enough support for the bath chair lift. It allows children who are cared for at home or in professional settings to bathe and maintain personal hygiene. The lowering mechanism makes it easier to put the child into the water and raise them up if the child is unable to sit or lie in the bathtub on their own.

Any use that goes above and beyond this does not comply with the intended use.

Indication / Patient Target Group / Intended Users

The product is suitable for children with limited control of their torso and / or reduced mobility of the joints in the lower extremities, and overall limited body control. Furthermore, it is used for children who have damage to their nervous system with neuromusculoskeletal and movement-related limitations.

Risks Associated with Use and Contraindications

If used properly, risks can be eliminated during use. There are no known contraindications.

Product Description

Scope of Delivery

Your product is delivered with the standard equipment and comes preassembled. A copy of the Instructions for Use is enclosed with the product.

Standard Equipment

Stainless steel frame; comfortable bolster; water-permeable mesh; straps; lowering system

Putting into Service

- Carefully remove the transport protection and the packaging.
- Keep the packaging material and small parts out of the reach of children to prevent swallowing or choking.

Teaching the User

The user is taught how to use this device by trained staff who will reference these Instructions for Use.

Adjustment Options / Features

Installing and removing the bath chair lift

To install the bath chair lift, simply place it on the sides of the bathtub with both hands.

Important: When placing the bath chair lift on the bathtub, make sure the lifting arm is secured. (Fig. 2)

To remove the bath chair lift, first secure the lifting arm. Then remove the bath chair lift from the sides of the bathtub with both hands.

Lowering and lifting

Lower the bath chair lift using the lifting lever. Release the lifting arm from the catch. (Fig. 2) Then swing the lifting arm over to lower the bath chair lift safely. Swing the lifting arm back to raise the bath chair lift. (Fig. 3)

Attention: Regularly check the tightness of the knurled screw locking. When the lifting arm is locked, the white safety plastic disk must always be positioned horizontally to the lifting arm tube.

Technical Data

Simplex

Seat width	32 cm / 12.6"
Bath width	54 - 64 cm / 21.3 - 25.8"
Max. lowering height	35 cm / 13.7"
Overall dimensions (l x w x h)	125 x 74 x 15 cm / 49.2 x 29.1 x 5.9"
Weight	8 kg / 17.6 lbs
Max. user weight	25 kg / 55.1 lbs

Material Specifications

high quality steel tubing coated with hardwearing powder enamel finish; reclining surface and belt are made of nylon

Cleaning, Maintenance, and Repair

Cleaning, Care Instructions, and Disinfection on a Daily Basis

Plastic Parts

The plastic parts on the product can be cleaned and disinfected with standard soap (neutral soap).

Metal Parts

The metal parts on the product can be cleaned and disinfected with standard soap (neutral soap).

Disinfection

Clean the plastic parts and metal parts before disinfecting.

Damp wipe all the plastic and metal parts on the product with disinfectant. Only water-based products should be used. When disinfecting note the directions from the manufacturer in question.

Maintenance and inspection by the distributor

Preparation

A copy of the Instructions for Use has to be enclosed with the product. If one is not available, request a copy from the distributor. Familiarise yourself with how the product works. If you are not familiar with the product, read the Instructions for Use before testing it. Clean the product before starting to test it. Please note the care instructions in the Instructions for Use and product-specific test instructions. If you have further questions, please contact the staff member you dealt with or Thomashilfen directly.

Service and repairs to the product should only be done by qualified staff. It is the user's responsibility to ensure that the prescribed inspections / maintenance are done regularly and in a timely fashion. If damage is apparent, the user has to actively inform and have the damage fixed immediately by qualified staff.

Maintenance Schedule – Annual Inspection

	Area	Function	Damage	Replace
1	Frame	👁 / ↻	👁 / ↻	
2	Adjustment options	👁 / ↻	👁 / ↻	
3	Reclining surface	👁 / ↻	👁 / ↻	
4	Belts	👁 / ↻	👁 / ↻	
5	Plastic parts	👁 / ↻	👁 / ↻	
6	Metal parts	👁 / ↻	👁 / ↻	
7	Screws and nuts	👁 / ↻	👁 / ↻	
8	Labels		👁 / ↻	
done by:			on:	
👁 = Inspection ↻ = Reuse ✓ = Test completed				

Spare Parts

Only original spare parts should be used for repairs. A list of the spare parts is available at www.thomashilfen.de. To receive the right spare parts or for possible technical recalls, we normally require the serial number / UDI-PI and the exact UDI-DI.

Removal / installation of spare parts should only be done by qualified staff! In the event of returns, please make sure the product is securely packaged for transportation.

Storage

Store the product in a safe place when it is not being used. Do not lay or put any heavy objects on the product while it is being stored. Protect the product from weather like rain, intense sunlight, and frost. If the product is wet, wipe the metal and plastic parts dry and let the padding dry before storing the product.

Disposal

If the product is no longer usable due to its condition or it has reached the end of its service life, it can be recycled at the local waste disposal company. Please refer to the local guidelines for recycling and disposal.

EU Declaration of Conformity

We hereby declare as the party solely responsible that this product and any versions and accessories comply with the basic regulations. The current EU Declaration of Conformity can be downloaded at www.thomashilfen.de or requested from us directly.

Warranty

Thomashilfen offers you the two-year warranty as required by law for this product. The warranty period starts when the item is handed over. The warranty includes all claims which impact function. Not included is damage which is the result of improper use (e.g. overloading) and natural wear and tear.

Period of Use

The product has a period of use of up to 6 years if used correctly and serviced regularly by a professional as per the information from the manufacturer. The product can be used beyond this point if it is in safe and technically sound condition. Time spent in storage at the distributor does not count towards the period of use.

The theoretical period of use does not equate to a guaranteed service life. It depends on the intensity of use and always undergoes an individual assessment by the distributor.

The period of use does not apply to wear parts like padding, covers, etc.

Service Life

The product has a service life of 8 years or 2 reuses (3 users) if the requirements stipulated for the period of use are met. The service life consists of the period of use and the storage times between reuses.

Reuse

The product is suitable to be reused. Before being passed on to the next user the product always has to be carefully cleaned and disinfected. The condition has to be checked by trained staff for any wear and damage. Any issues have to be remedied to ensure the continued safe use of the product. A copy of the Instructions for Use is a component of the product and has to be passed on to the new user.

Distributor

NRS Healthcare

Sherwood House, Cartwright Way

Forest Business Park

Leicestershire LE67 1UB

Phone: +44 345 121 8111

Fax: +44 845 121 8112

customerservice@nrshealthcare.co.uk

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